

KEMIS.



SINGLE USE
SURGICAL KNIVES





Kemis[®] .

Intended purpose:

Knives are intended for cutting soft tissues.

Indications:

The knives of the Kemis range are intended to be used in case of nerve, tendons or muscle compression.

Kemis knives range.

The Kemis® knives range currently consists of two different knives.

Each of them have been specifically designed to treat a large number of applications:


- The Kemis H3
- The Kemis H1



Kemis H3




Kemis H1

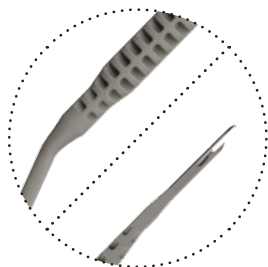


ERGONOMICALLY ENHANCED INSTRUMENTS

- Ergonomic design, upper and lower curves
- End tip designed to protect the blade

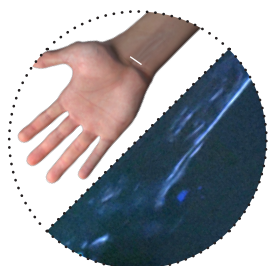

SURGICAL INFORMATION

- Small incision (7 to 15 mm for Kemis H3 and 3 to 5 mm for Kemis H1)
- Possibility of visual monitoring by ultrasound

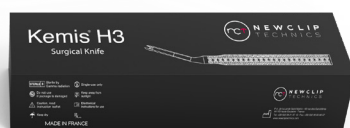

SINGLE USE STERILE KNIVES


- Ready to use for surgery
- Sterile to minimize the risks of contamination.

Technical features.

KEMIS H3

APPLICATIONS

Please find below a non-exhaustive* list of applications that the Kemis H3 knife can be used to treat:

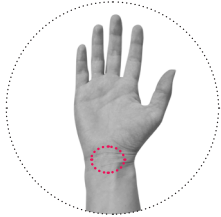


Carpal tunnel syndrome




**Tennis elbow
(lateral epicondylitis)**



**Compartment syndrome
(upper and lower limb)**

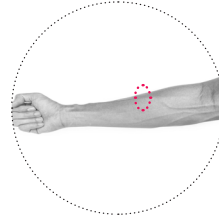


**Lacertus fibrosus
syndrome**

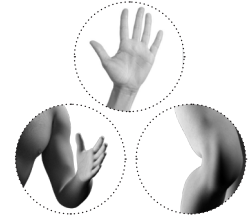

**Spasticity
(treatment by tenotomy)**


TECHNICAL FEATURES

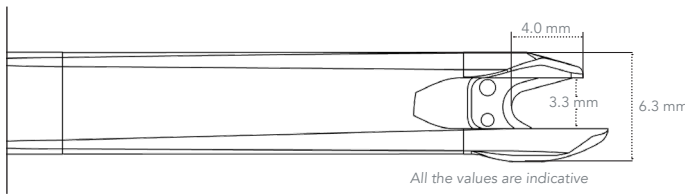


KEMIS H1

APPLICATIONS


Please find below a non-exhaustive* list of applications that the Kemis H1 knife can be used to treat:

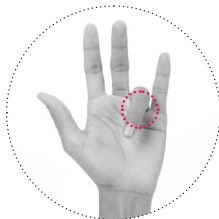

Trigger finger

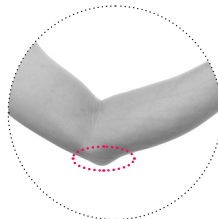


Ulna nerve entrapment

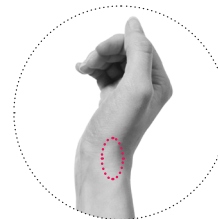

De Quervain's tenosynovitis

TECHNICAL FEATURES

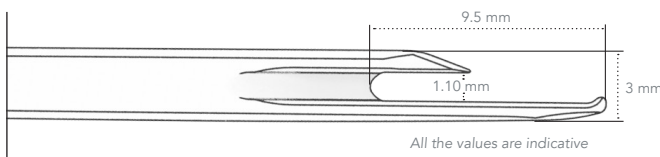


*This instrument can also be used for other canalar syndromes.

Surgical technique.

KEMIS H3 - CARPAL TUNNEL RELEASE

Please note that this is one of the applications for the Kemis H3 knife, it can also be used for numerous other applications (see page 3). The technique presented below is one of the surgical techniques possible. The choice is left to the surgeon's discretion.


This surgical technique includes the use of ultrasound, which gives a real-time control of the nerves, tendons and arteries during the whole procedure. This surgery can also be performed without ultrasound.

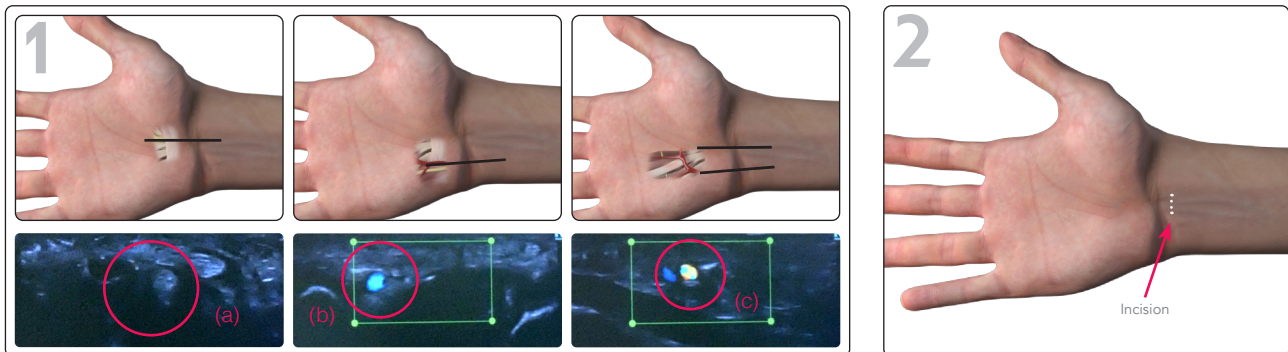


Using ultrasound, visualize: the median nerve **(a)**, the ulnar artery **(b)** and the palmar arterial arch **(c)**.

Position the wrist in an extension position and make an incision at the fold of the wrist by keeping in reference the two lines for the medial (median nerve) and the lateral (ulnar artery) limits.

Then, detach the surrounding tissues with scissors or with small forceps.

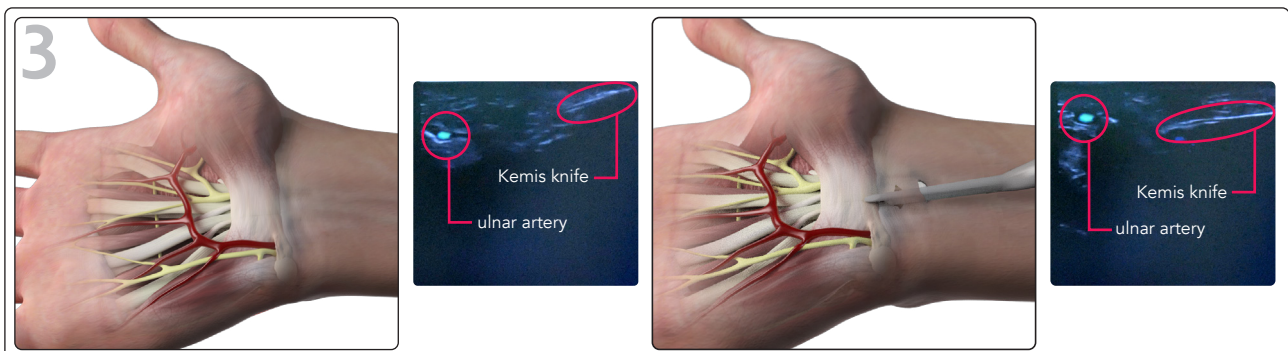

The Kemis H3 knife is applied to the proximal lip of the ligament, continue inserting the knife while applying continuous pressure until the blade cuts through the ligament. During the procedure, maintain dynamic control of the movement. Check if the anterior carpal ligament cut is complete using the ultrasound system and scissors or small forceps.

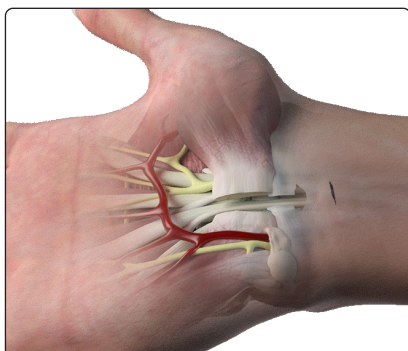


**FINAL
RESULT.**


KEMIS H1 - TRIGGER FINGER

Please note that this is one of the applications for the Kemis H1 knife, it can also be used for numerous other applications (see page 3). The technique presented below is one of the surgical techniques possible. The choice is left to the surgeon's discretion.


This surgical technique includes the use of ultrasound, which gives a real-time control of the nerves, tendons and arteries during the whole procedure. This surgery can also be performed without ultrasound.

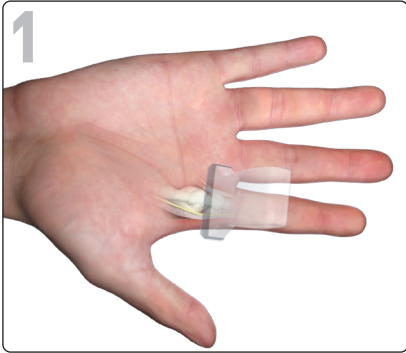


1 Visualize the tendon nodule by using ultrasound. Once the area has been identified, mark it and make a 3 to 5 mm incision.

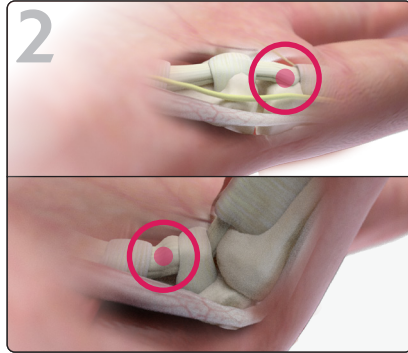



2 Dynamic control of the nodule by moving the finger.

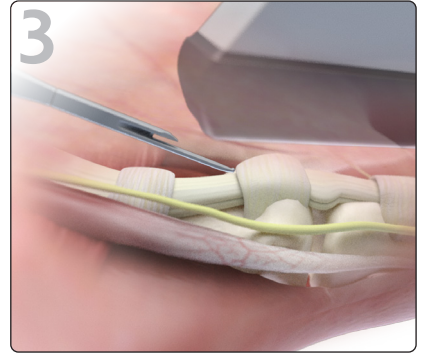

3 Insert the Kemis H1 in the sheath, proximal to the A1 pulley. The insertion can be controlled under ultrasound.

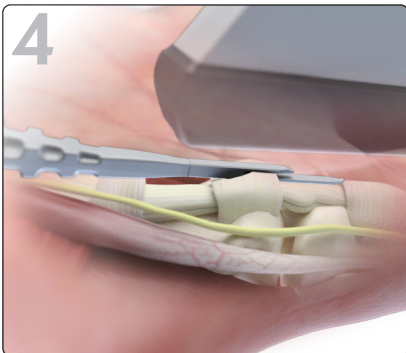

4 Continue inserting the Kemis H1 while applying continuous pressure until the blade cuts through the A1 pulley.

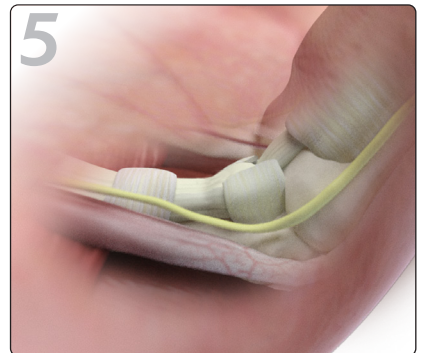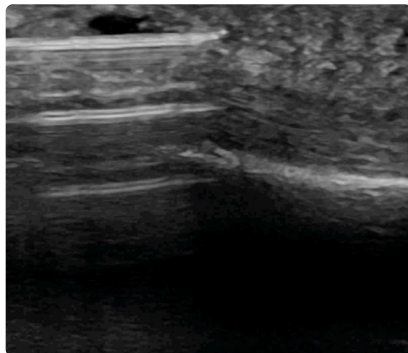

5 Test the release using the ultrasound visualization by moving the finger.

RETROGRADE TECHNIQUE



Depending on the surgeon's habit, the cut can also be performed using a retrograde technique. The surgical techniques follow the same steps as explained previously.

References.

Kemis H1

Ref.	Description	Qty
ANC1103	KEMIS H1 knife	5



Kemis H3

Ref.	Description	Qty
ANC209	KEMIS H3 knife	5



This information is intended to demonstrate the Newclip Technics portfolio of medical devices. Always refer to the package insert, product label and/or user instructions including cleaning and sterilization before using any Newclip Technics product. These products must be handled and/or implanted by trained and qualified staff who have read the instructions before use. A surgeon must always rely on her or his own professional clinical judgement when deciding whether to use a particular product when treating a particular patient. Product availability is subject to the regulatory or medical practices that govern individual markets. Please contact your Newclip Technics representative if you have questions about the availability of Newclip Technics products in your area.

Manufacturer: Newclip Technics - Brochure EN+US - Kemis - Ed.11 - 05/2026 - Medical devices: class IIa - CE1639 SGS BE - US Class: I - Read labelling and instructions before the use of Newclip Technics medical devices. These products must be handled and/or implanted by trained and qualified staff who have read the instructions before use. Non-contractual pictures.
Newclip Technics - 45 rue des Garottières - 44115 Haute Goulaine, France. Our subsidiaries: Newclip USA - Newclip Australia - Newclip GMBH - Newclip Japan - Newclip Iberia - Newclip Belgium.

newcliptechnics.com

